

# Cardboard Box

BROWN BOX – RED BAG



1

Turn over and seal bottom flaps with tape




2

Place liner into box and fold over the edges. Fill box with regulated medical waste. **No loose sharps are permitted outside of sharp collectors.**



3

Seal all liners by twisting and hand-tying into a single knot.

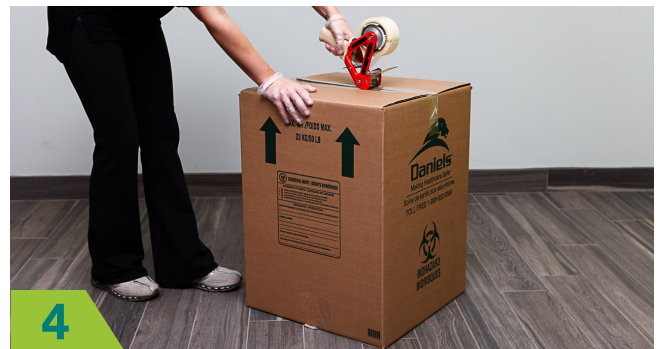

4

Close lid by tapping the flaps down. Securely store in waste collection area. **Liners should not be showing once the flaps have been taped.**



5

Label the box correctly with the waste type packaged within.



6

Contact your local Daniels service location for a pick-up of your container.

## Small Cardboard Box

Dimensions: 14" x 14" x 21"  
Max Weight: 16kg

## Big Cardboard Box

Dimensions: 18" x 18" x 24"  
Max Weight: 23kg

USN221027

WWW.DANIELSHEALTH.COM